Anuform®

Instruction Manual – GB

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Manual de Instrucciones – ES

Handleiding – NL

Brukerveiledning – NOR

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inner strength, outer confidence

Indications for Use:

For use as an intra-anal probe in males and females and a small vaginal probe in females. For use only with muscle stimulation and EMG biofeedback equipment approved to EN60601-01 in the treatment of 'Stress', 'Urge' and 'Mixed' incontinence and faecal incontinence.

FOR SINGLE PATIENT USE – may be re-used by the same patient.

FOR SINGLE ORIFICE USE – not for both vaginal and anal use.

Use as directed by your medical adviser.

Contraindications & Precautions:**DO NOT USE:**

With electrical stimulation if fitted with a cardiac pacemaker • If pregnant

- If the package seal is not intact
- If an infection or tissue damage or recent surgical scar is present
- Vaginally during menstruation
- With electrical stimulation in patients with a history of carcinoma at the site of stimulation
- If an irritation or discomfort occurs and seek medical guidance.

CAUTION IS ADVISED:

When using electrical stimulation on patients with altered or reduced tissue sensation • During electrical stimulation if the user has a history of epilepsy - seek informed medical guidance • When using the ANUFORM® with equipment capable of delivering outputs in excess of 2mA/cm². If in doubt contact the equipment manufacturer or your medical adviser.

Cleaning:

- Wash ANUFORM® in warm soapy water.
- Rinse in clean water and dry.
- Repeat after each use and dry thoroughly before storing in its plastic bag.
- DO NOT use boiling water.

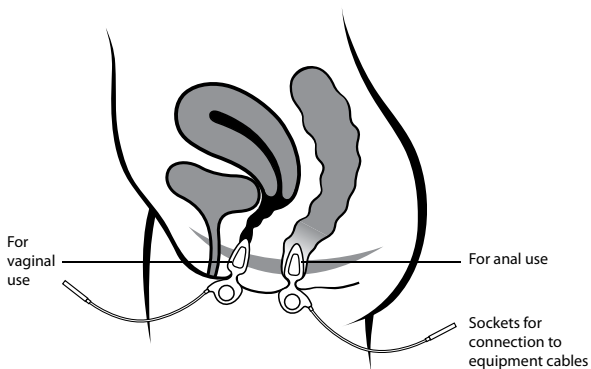
Disposal:

Clean thoroughly and dispose of according to local regulations.

Indications for Use:

IMPORTANT:

- Before first and subsequent use wash and dry as directed.
- Apply light coating of KY Jelly or similar lubricating gel to tip and metal surfaces of probe to aid insertion and provide good electrode conductivity.
- **Anal use:** Insert into anus while 'bearing down' as if passing stool. 'Ring' part of Anuform should remain external to the anus at all times.
- **Vaginal Use:** Insert into vagina in an 'East/West' position, i.e. with the metal electrode surfaces facing towards the hips. Insert until body of Anuform is within the vaginal vault. 'Ring' part of probe should remain externally within labia.
- Ensure equipment is turned off before connecting to probe cables.
- Use as directed by your medical adviser. After use, turn off equipment, disconnect cables. Do not pull on cables to remove probe.
- Wash, dry and store as instructed on previous page.

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